


# HEINE BETA X Ophthalmoscope

## THE MOST IMPORTANT FUNCTIONS AT A GLANCE

To ensure optimal battery performance and longevity,  
fully charge your HEINE BETA X Ophthalmoscope  
to 100 % before its first use.

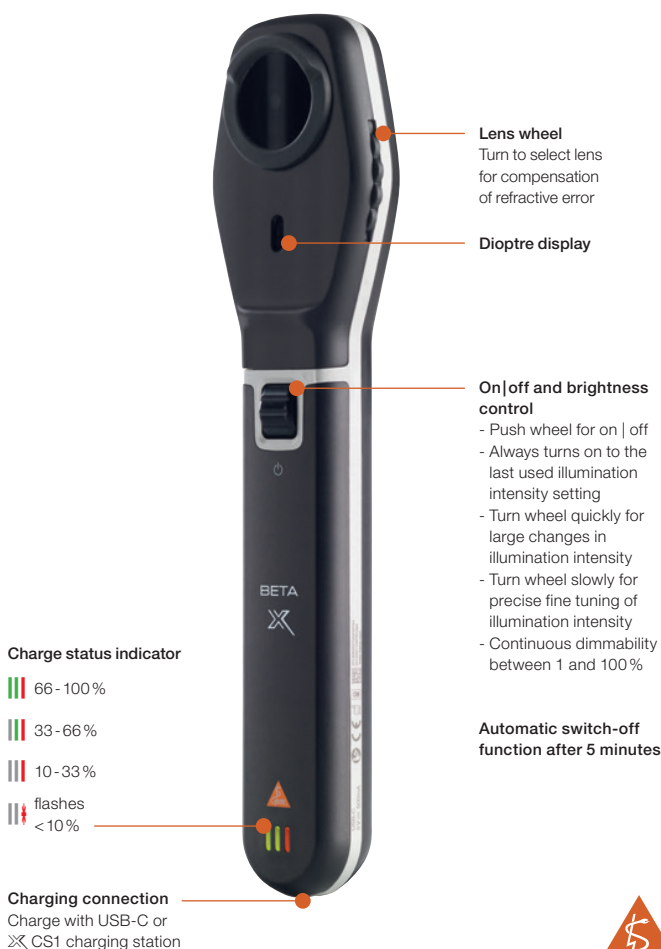

QUALITY  
MADE IN GERMANY



# HEINE BETA X Ophthalmoscope

## HEINE

### focusCONTROL

Switch between examination of the posterior (F = fundus) and anterior segment (C = cornea) of the eye

### Filter selection

-  red-free filter
-  without filter
-  polarisation filter





### Aperture wheel

7 different apertures which can also be used with red-free filter or with polarisation filter



For quick and easy results in fundus examinations choose the settings marked with a white dot:

-  Filter: polarisation filter
-  Aperture: large spot
- F focusCONTROL: fundus



More information  
on [heine.com](http://heine.com)

